UK

INTRODUCTION

Please read these instructions carefully before using your contact lenses. Seek advice from your eye care practitioner for recommendations on lens wearing, cleaning and replacement regime.

INDICATIONS AND INTENDED USE

MAXVUE VISION Contact Lenses are intended for the optical correction of refractive error, myopia (nearsightedness), hyperopia (farsightedness) (long-sightedness) and/or astigmatism in persons with non-diseased eyes. The lens is also used to enhance or alter the apparent colour of the eye (for Coloured Lenses).

Your Eye Care Professional may prescribe the lenses either for single use disposable wear or frequent/planned replacement wear with cleaning, disinfection and scheduled replacement. When prescribed for frequent/planned replacement wear, you may clean and disinfect the lenses using a chemical disinfection system.

PRECAUTION FOR USE

Handling precaution

1. Do not use if the sterile blister or vial is opened or damaged
2. Always wash your hands with mild soap and water for 20 seconds before handling your lenses.
3. Do not get cosmetics, lotions, soaps, creams, deodorants or sprays in your eyes or on your lenses. It is best to put on your lenses before putting on makeup. Water-based cosmetics are less likely to damage lenses than oil-based products.
4. Do not touch your contact lenses with your fingers or hands if they are not completely clean. Tiny lens scratches may occur, causing unclear vision and/or injury to your eyes.
5. Carefully follow the handling, insertion, removal and wearing instructions in this instruction and as prescribed by your Eye Doctor.
6. Never use tweezers or other tools to remove your lenses from the lens container unless specifically indicated for that use. Pour the lens and the packing solution into the hand.
7. Do not touch the lens with your fingernails.
8. Clean, rinse and disinfect your lenses each time you remove them.
9. Always handle the same lens first to avoid mixing up the right and left lenses.
10. Do not rinse your lenses or lens case with tap water or allow lenses to come into contact with non-sterile water or solution.
11. Remember to handle your lenses gently.

Lens wearing precaution

1. You should remove your lenses immediately if your eyes become red, itchy, sore, or irritated.
2. Never wear your lenses beyond the amount of time recommended by your Eye Doctor.
3. If aerosol (spray) products, such as hair spray, are used while wearing lenses, exercise caution and keep your eyes closed until the spray has settled.
4. Avoid all harmful or irritating vapors and fumes while wearing lenses.
5. Ask your Eye Doctor about wearing contact lenses during sports activities, especially swimming and other water sports. Wearing contact lenses while water during swimming or while in a hot tub may increase the risk of eye infection from germs.
6. Soft contact lenses can change the performance of the lens if improperly maintained. If you experience any abnormal symptoms such as blurred vision when you wear the lens, stop wearing the lens, check with your ophthalmologist and check the performance and condition of the lens before deciding whether to wear it.

Solution Precautions:

1. Never use solution not recommended for conventional hard contact lenses only.
3. Always use fresh lens care products and replace them before the expiration date.
4. Always follow the directions in the package inserts for the use of contact lens solutions.
5. Use only a chemical (not heat) lens care system. Use of heat (thermal) care systems can damage your contact lenses.
6. When you use sterile solutions that do not contain preservatives, they should be thrown out after the time specified in the directions.
7. Do not use saliva or anything other than the recommended solutions for lubricating or wetting lenses.
8. Always keep your lenses completely covered by the recommended storage solution when the lenses are not being worn (stored). Extended periods of drying will make it harder for the lens to become wet again. Follow the lens care directions for "Care for a Dried Out (Dehydrated) Lens" if the lens surface becomes dried-out.
9. If the lens sticks (stops moving) on your eye, follow the recommended directions in "Care for a Sticking Lens". The lens should move freely on your eye for the continued health of your eye. If non-movement of the lens continues, you should immediately consult your Eye Doctor.

1. Always contact your Eye Doctor before using any medicine in your eyes.
2. Be aware that certain medications, such as antihistamines, decongestants, diuretics, muscle relaxants, tranquilizers and the tip or other eye sickness may cause dryness of the eye, increased lens awareness (feeling of the lens in the eye) or blurred vision. Always inform your Eye Doctor if you experience any problems with your lenses while taking such medications. Depending on the severity, your Eye Doctor may prescribe the use of lubricating drops that are indicated for use with soft contact lenses or may recommend that you stop wearing contact lenses while you are using these medications.
3. Be aware that if you use oral contraceptives (birth control pills), you could develop changes in vision or comfort when wearing contact lenses.

recommended by the lens care manufacturer or your Eye Care Professional.

Other precaution to discuss with your Eye Doctor

1. Always contact your Eye Doctor before using any medicine in your eyes.
2. Be aware that certain medications, such as antihistamines, decongestants, diuretics, muscle relaxants, tranquilizers and the tip or other eye sickness may cause dryness of the eye, increased lens awareness (feeling of the lens in the eye) or blurred vision. Always inform your Eye Doctor if you experience any problems with your lenses while taking such medications. Depending on the severity, your Eye Doctor may prescribe the use of lubricating drops that are indicated for use with soft contact lenses or may recommend that you stop wearing contact lenses while you are using these medications.
3. Be aware that if you use oral contraceptives (birth control pills), you could develop changes in vision or comfort when wearing contact lenses.

LENS HANDLING AND INSERTION

Inserting the lens into your eyes

1. You may want to rinse your lenses in a multi-purpose solution before you put them on. Put your lens on the lens or your index finger. Be sure that the lens is not inside out (the edges should turn up and not out).
2. Look straight ahead. Pull down your lower eyelid with your middle finger of the same hand.
4. Pull the top lid up with the index finger of the other hand and look up.
5. Gently place the lens on the lower white of the eye.
6. Remove your index finger and release your eye. The lens will centre itself.
7. Repeat for the other lens.

Centering the lens

A lens, which is on the cornea, will very rarely move onto the white part of the eye during wear. This, however, can occur if insertion and removal packaging is used incorrectly. To center a lens, follow either of these procedures:

1. Close your eyelids and gently massage the lens into place through the closed lids. OR
2. Gently move the off-centered lens onto the cornea while the eye is opened using finger pressure on the edge of the upper lid or lower lid.

Removing the lens from your eye

1. Look up and pull the lower lid down with your middle finger.
2. Then place your index finger on the lower edge of the lens, and slide the lens down to the white of your eye.
3. Gently move the lens lightly in between your index finger and thumb and remove gently.
4. Repeat with the other eye.
5. If the edges of your lens stick together, place a few drops of saline or multi-purpose solution on the lens and rub gently until the edges separate.

Care for a Dried Out (Dehydrated) Lens

If a soft contact lens is exposed to air for prolonged periods while off the eye, it may become brittle and can be easily broken. If this happens, throw away the lens and use a new fresh one.

Care for a Sticking (Non-moving) Lens

If a lens sticks (stops moving) on your eye, apply a few drops of the recommended lubricating solution. You should wait until the lens begins to move freely on the eye before removing it. If non-movement of the lens continues, you should immediately consult your Eye Doctor.

Cleaning & Chemical Disinfecting

1. Follow the instructions provided in the cleaning and packaging or recommend by your Eye Doctor.
2. Always clean the same lens first to avoid mix-ups.
3. After cleaning, disinfect the lenses using the system recommended by your Eye Doctor.
4. Put the lens into the correct chamber of the lens storage case.
5. Do not heat the disinfection solution and lenses.

Caution: Lenses that are chemically disinfected may absorb ingredients from the disinfecting solution that may be irritating to the eyes. A thorough rinse in fresh sterile saline (or another recommended solution) prior to placement in the eye should reduce the potential for irritation.

Storage

1. The lens should be stored at room temperature (4°C to 35 °C).
2. To store your lenses, first disinfect them and then leave them in the closed/unopened case until you are ready to wear them.
3. Always keep your lenses completely covered by a recommended disinfecting solution when the lenses are not being worn.
4. After removing your lenses from the lens case, immediately rinse the lens storage case with solutions recommended by your Eye Doctor or the lens care manufacturer, then allow the lens case to air dry.
5. When the case is used again, refill it with fresh

storage solution. Replace lens case at regular intervals.

CONTRAINDICATIONS

Maxvue Vision's contact lenses are not recommended for patients who experience symptoms such as

1. Eye soreness
2. Red eyes
3. Any active corneal infection (bacterial, fungal, protozoal or viral)
4. Eye injury to the eyelid or cornea which may be aggravated by lenses.
5. Severely dry eyes
6. Irregular eye sensitivity
7. Eye infection or eye inflammation
8. If any other ingredient used in the lens is making or in a solution which is to be used to care for your contact lens

WARNINGS

1. Remove lenses daily to clean, rinse & disinfect. Fill your contact lens case with multi-purpose solution and soak for at least 4 hours before re-insertion.
2. Wearing of the contact lenses while sleeping or extended wear increases the risk of infection & permanent damage to vision.
3. Infection, with possible permanent damage to vision, could result from the failure to strictly follow recommended directions for use. Lens care procedures including lens cases.
4. Always store contact lenses in a disinfecting or saline solution. Never leave the lenses dry out.
5. If lenses are not used in a period more than the labelled replacement schedule, dispose and use a fresh pair.
6. The over use of ulcerative keratitis may be reduced by carefully following directions for lens care, including cleaning the lens case. The risk of ulcerative keratitis among contact lens users who smoke is greater than among non-smokers.

POTENTIAL ADVERSE EFFECTS

1. Due to the following problems may occur when wearing contact lenses. Please consult with your eye doctor:
1. Your eyes may burn, sting and/or itch.
2. There may be less comfort than when the lens was first placed on the eye.
3. There may be a feeling of something in your eye.
4. There may be the potential for some temporary harm due to peripheral irritations, peripheral corneal ulcers and corneal erosion.
5. There may be excessive watering, unusual eye secretions or redness of your eye.
6. Poor vision, blurred vision, rainbows or halos around objects, sensitivity to light (photophobia) or dry eyes may also occur if your lenses are worn continuously or for too long a time.

DISPOSAL OF LENS AFTER USE

Dispose the contact lens according to general household waste disposal.

WEARING SCHEDULE

The lenses are indicated for daily wear (less than 24 hours, while awake). You may refer to product packaging for replacement information. The eye care practitioner is encouraged to determine an appropriate lens replacement schedule based on the response of the patients.

Wearing schedule (this information is for first time users)

Day	Wearing Time (Hours)
1	4
2	5
3	6
4	7
5	8
6	9
7	10
8	11
9	12
10	Within 12 hours

Studies have not completed to show that Maxvue Vision Soft contact Lenses are safe to wear during sleep.

SYMBOLS EXPLANATION

LOT	Lot Number	CESAM	Chemical disinfectant cleaning system
DIA	Diometer	B.C	Burn Case
D	Back View Lens	C.A.L	Chemical disinfectant cleaning system
EXP	Expiration Date	M	Manufacturer
Caution symbol		Easy symbol	
Do not use symbol		Do not use symbol	
Keep away from heat symbol		Temperature control	
Use lenses the same way as the lens of the eye		Single use only	
MD	Medical Device		
UDI	Unique Device Identifier		

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